




Measure in cms around your arm, approx 4cms above the wound dressing, plaster cast or affected area.

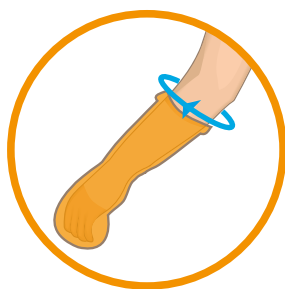

For example the blue arrow indicates where the measurements should be taken. Needs to be on the fleshy part of your arm.



Arm DryLimb covers

Waterproof wound dressing and cast protectors



Shower Bath Swim



Below elbow

Short Arm


Dressed wounds & splints below elbow	
S = 30cm SA/S/30	<input type="checkbox"/>
M = 40cm SA/M/40	<input type="checkbox"/>
L = 50cm SA/L/50	<input type="checkbox"/>



Fingers/hand/wrist

Medium Arm

Dressed wounds & plaster casts to elbow	
S = 30cm MA/S/30	<input type="checkbox"/>
M = 40cm MA/M/40	<input type="checkbox"/>
L = 50cm MA/L/50	<input type="checkbox"/>

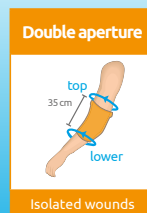

Double aperture

Isolated wounds

Arm wounds & PICC lines	
S = 40cm CC/S/40/40	<input type="checkbox"/>
M = 50cm CC/M/50/40	<input type="checkbox"/>
L = 60cm CC/L/60/50	<input type="checkbox"/>



Fingers/hand/wrist

Full Arm

Dressed wounds & plaster casts below shoulder	
S = 40cm FA/S/40	<input type="checkbox"/>
M = 50cm FA/M/50	<input type="checkbox"/>
L = 60cm FA/L/60	<input type="checkbox"/>

Take a top and lower measurement.
i.e. top = 40cm and lower = 40cm
order CC/S/40/40.



Re-usable Latex-free Soft & flexible

QOL Recommends

As our arm products do not have a formed foot, they are ideal for amputees. We also recommend our arm products for leg injuries on babies and small children.



Please see video on our website.

S = Small | M = Medium | L = Large
all our sizes are measured in cms

QOL.Ltd

www.buddycover.com

e. sales@buddycover.com

t. UK Freephone: 0800 2700074

How to use your buddy®

See how to measure overleaf

Gently stretch open the seal. Slip your **buddy®** over your hand and pull up gently.



Pull the corners down slightly creating a smooth seal around your arm.

You are ready to shower!



For swimming and bathing:
Gently pull the corner to open the seal as you lower your arm into the water. Let go of the seal just before you fully submerge your arm.



You can swim, bath and shower with confidence.

Your buddy® won't let water in.

- Avoid sharp objects whilst wearing/handling your **buddy®** product
- Supervise children with **buddy®** products at all times. This is not a toy
- Do **NOT** use bleach, to clean - rinse and leave to dry naturally
- These products can be used inside out.

Also available for legs.

