



Measure in cms around your leg, approx 4cms above the wound dressing, plaster cast or affected area.



For example the blue arrow indicates where the measurements should be taken. Needs to be on the fleshy part of your leg.

For plaster casts below the knee use a long leg buddy®.

Order Code: ML/S30 = medium leg / small circumference 30cm.



Leg DryLimb covers

Waterproof wound dressing and cast protectors



Shower Bath Swim



Dressed wounds	
S = 30cm SL/S/30	<input type="checkbox"/>
M = 40cm SL/M/40	<input type="checkbox"/>
L = 50cm SL/L/50	<input type="checkbox"/>



Dressed wounds	
S = 30cm ML/S/30	<input type="checkbox"/>
M = 40cm ML/M/40	<input type="checkbox"/>
L = 50cm ML/L/50	<input type="checkbox"/>

* Available in XL = 60cm



Dressed wounds & plaster casts below & above knee	
S = 50cm 3Q/S/50	<input type="checkbox"/>
M = 70cm 3Q/M/70	<input type="checkbox"/>
L = 90cm 3Q/L/90	<input type="checkbox"/>



Dressed wounds & plaster casts below & above knee	
S = 50cm LL/S/50	<input type="checkbox"/>
M = 70cm LL/M/70	<input type="checkbox"/>
L = 90cm LL/L/90	<input type="checkbox"/>

* Available in XXS = 30cm & XS = 40cm

Take a top and lower measurement. i.e. top = 40cm and lower = 40cm order CC/S/40/40.



Dressed wounds & plaster casts below & above knee	
S = 40/40cm CC/S/40/40	<input type="checkbox"/>
M = 50/40cm CC/M/50/40	<input type="checkbox"/>
L = 60/50cm CC/L/60/50	<input type="checkbox"/>

S = Small | M = Medium | L = Large
all our sizes are measured in cms

Re-usable Latex-free Soft & flexible

QOL.Ltd

www.buddycover.com

e. sales@buddycover.com

t. UK Freephone: 0800 2700074

How to use your buddy®


See how to measure overleaf

Gently stretch open the seal. Slip your **buddy®** over your foot and pull up gently.



Pull the corners down slightly creating a smooth seal around your leg.

You are ready to shower!

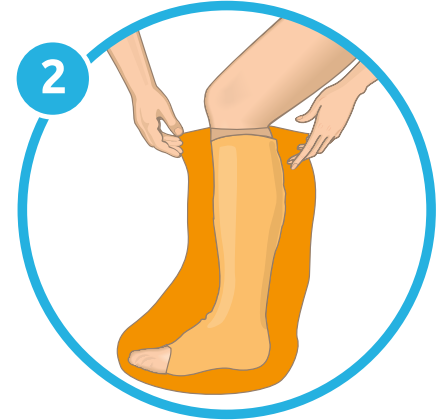

For swimming and bathing:
Gently pull the corner to open the seal as you lower your leg into the water. Let go of the seal just before you fully submerge your leg.



You can swim, bath and shower with confidence.

Your buddy® won't let water in.

- Avoid sharp objects whilst wearing/handling your **buddy®** product
- Supervise children with **buddy®** products at all times. This is not a toy
- Do **NOT** use bleach, to clean - rinse and leave to dry naturally
- These products can be used inside out.

Also available for arms.

